



**CarbonBased, LLC**™

*Engineering Orthotic Solutions*

169 Commack Road – Suite H #158

Commack, NY 11725

T: 888.760.7223 | E: [info@braceforpectus.com](mailto:info@braceforpectus.com)

[www.braceforpectus.com](http://www.braceforpectus.com)

# P4COft

## Adjusting The Sternal Pad Location

- Remove the pad cover
  
- Remove the sternal pad to expose the truss nuts
  
- Unscrew both the Philips head screws from the carbon fiber bar.
  
- Detach the sternal plate from carbon fiber bar and move to desired location left or right.
  
- Relocate truss nuts as necessary to align with holes in carbon fiber bar.
  
- Press sternal plate back in place against carbon fiber bar.
  
- Secure sternal plate with Philips head screws and star washers.
  
- Reapply both sternal pad and pad cover.